

## Aluminum Finger Splint (Protector)

### Product description

Formedica's aluminum splints allow to immobilize temporarily fingers and wrist in case of sprain or fracture. Their aluminum side provides the necessary rigidity and their foam side ensures patient's comfort.

The four-prong design allows fast and easy application. Used to protect cuts, crushed fingers and injuries.



### Characteristics

- Made of malleable aluminum, padded with foam
- Radiolucent
- Easy removal for cleaning or examination purposes
- Splint model must be chosen according to the type of fracture and degree of immobilization required

### Available sizes

CODE	CODE DOZEN	LENGHT	SIZE
9176	129176	5.7 cm (2.25")	SMALL
9177	129177	7.3 cm (2.90")	MEDIUM
9178	129178	8.3 cm (3.25")	LARGE

## Aluminum Finger Splint (Baseball)

### Product description

Formedica's aluminum splints allow to immobilize temporarily fingers and wrist in case of sprain or fracture. Their aluminum side provides the necessary rigidity and their foam side ensures patient's comfort.



Immobilizes and protects distal phalanx fractures.

### Characteristics

- Made of malleable aluminum, padded with foam
- Radiolucent
- Easy removal for cleaning or examination purposes
- Splint model must be chosen according to the type of fracture and degree of immobilization required

### Available sizes

CODE	CODE DOZEN	LENGHT	SIZE
9186	129186	13.4 cm (5.25")	SMALL
9187	129187	14.7 cm (5.75")	MEDIUM
9188	129188	15.9 cm (6.25")	LARGE

## Aluminum Finger Splint Kit

### Product description

Formedica's aluminum splints allow to immobilize temporarily fingers and wrist in case of sprain or fracture. Their aluminum side provides the necessary rigidity and their foam side ensures patient's comfort.

Sold in a plastic case.

### Characteristics

- Made of malleable aluminum, padded with foam
- Radiolucent
- Easy removal for cleaning or examination purposes

\* Contents of kit 9195:

- 3 units each of 9146, 9147, 9148, 9157, 9158, 9167, 9168, 9169, 9176, 9177, 9178, 9186, 9187 and 9188
- 6 units of 9156

### Available sizes

CODE	PACKAGING	SIZE
9195	1 BOX OF 51 SPLINTS	MULTIPLE



## Wrist and Ankle Restraints

### Product description

Recommended to protect tubing or dressings of confused and agitated patients. Can be used on wrist or ankle.

### Characteristics

- Terry cloth, synthetic fur (kodel) or felt lining (depending on model)
- One size for adults
- Soft and resistant
- Allow “triangulation,” which limits the patient’s range of motion



### Available sizes

CODE	INTERIOR LINING	COLOUR	SIZE
7635	BLACK TERRY	STRAPS : BLACK EXTERIOR : PATTERN FOR KIDS	BABY
7636	BLACK TERRY	STRAPS : BLACK EXTERIOR : PATTERN FOR KIDS	CHILD
7638	BLACK TERRY	STRAPS : BLACK EXTERIOR : GREY	ADULT
7647	WHITE KODEL	STRAPS : WHITE EXTERIOR : WHITE	ADULT
7651	WHITE FELT	STRAPS : WHITE EXTERIOR : WHITE	ADULT

### Adjustment :

1. Check if restraints are in good condition.
2. Bring the straps to the bedside of the user.
3. Place the restraint on the bed and put gently the user’s wrists or ankle on the woolly felt.
4. Cross the ends of the belts and make sure that :
  - The first belt is threaded in the slot of the opposite side.
  - The second belt passes behind the restriction and slips into the hemming.
5. Interlock the two extremities and make sure it is not too tight.
6. Adjust the restraint belts firmly.
7. Fix it to the bed or the chair.
8. Check the comfort and the good body alignment of the user.

**Specifications :**

- Make sure the belts do not touch the skin, in order to avoid rubbing.
- Leave two finger space between user's wrist or ankle and the restraint.
- Make a simple knot and a half loop, then fix the restraint under the mattress.

\* PLEASE NOTE THAT THIS INSTRUCTION SHEET IS GIVEN TO YOU FOR INFORMATIONAL PURPOSES ONLY AND IT IS THE RESPONSIBILITY OF THE ESTABLISHMENT TO ENSURE THAT THE METHOD OF USE OF THE PRODUCT IS SUITABLE FOR EACH SPECIFIC CIRCUMSTANCES OF THE USER.



## Criss-Crossed Safety Vest

### Product description

This Criss-Crossed Safety Vest prevent the patient from leaning forward while allowing limited freedom of movement.

### Characteristics

- **Important** : The safety vest must be criss-crossed on the chest area. Never put it on backwards.
- Shoulder loops prevent the patient from leaning forward
- Polyester mesh
- 2 extra-long nylon web straps
- Adult sizes



### Available sizes

CODE	COLOUR	SIZE
7907	WHITE WITH PINK PIPING	SMALL
7908	WHITE WITH GREEN PIPING	MEDIUM
7909	WHITE WITH BLUE PIPING	LARGE

### Adjustment

1. Install the vest on the patient, the part with the FORMEDICA® label on the back of the patient. Close the zipper on the back.
2. Place the patient in the wheelchair.
3. Wrap the straps around the armrests of the wheelchair.
4. Fasten securely and ensure that the ends do not reduce or impede the movement of the person's chair. Let the thickness of a hand between the waist of the patient and the contention.
5. Ensure that the patient is in a comfortable position. Check the condition of the skin at the waist and hips every hour.

**Warning:** Use only while seated.

\* PLEASE NOTE THAT THIS INSTRUCTION SHEET IS GIVEN TO YOU FOR INFORMATIONAL PURPOSES ONLY AND IT IS THE RESPONSIBILITY OF THE ESTABLISHMENT TO ENSURE THAT THE METHOD OF USE OF THE PRODUCT IS SUITABLE FOR EACH SPECIFIC CIRCUMSTANCES OF THE USER.

## Poncho Safety Vest

### Product description

This Poncho Safety Vest prevent the patient from leaning forward while allowing limited freedom of movement.

### Characteristics

- Slide easily over the head
- Polyester mesh
- 4 extra-long nylon web straps
- Extra-wide 1" straps
- Belt straps guarantee a snug comfortable fit
- Adult sizes



### Available sizes

CODE	COLOUR	SIZE
7915	WHITE WITH PINK PIPING	SMALL
7916	WHITE WITH GREEN PIPING	MEDIUM
7917	WHITE WITH BLUE PIPING	LARGE

### Adjustment

1. Install the vest on the patient, the part with the FORMEDICA® label on the back of the patient.
2. Pass the belts through the slots. Reach through the third slot and slide the belt against the side. Slide one of the two belts through the slots leaving the other in front for locking. Tie a knot with the straps and loops on each side.
3. Place the patient in the wheelchair or in bed.
4. Wrap the straps around the armrests of the wheelchair or bed rails.
5. Fasten securely and ensure that the ends do not reduce or impede the movement of the person. Let the thickness of a hand between the waist of the patient and the contention.
6. Ensure that the patient is in a comfortable position. Check the condition of the skin at the waist and hips every hour.

**Warning:** Use only while seated.

\* PLEASE NOTE THAT THIS INSTRUCTION SHEET IS GIVEN TO YOU FOR INFORMATIONAL PURPOSES ONLY AND IT IS THE RESPONSIBILITY OF THE ESTABLISHMENT TO ENSURE THAT THE METHOD OF USE OF THE PRODUCT IS SUITABLE FOR EACH SPECIFIC CIRCUMSTANCES OF THE USER.

## Pelvic Belt

### Product description

The Pelvic Belt prevent patients from sliding forward.

### Characteristics

- Available in two types of material: polyester mesh or soft polyurethane foam laminated with cotton and a terry cloth interior
- 4 extra-long nylon web traps



### Available sizes

CODE	MATERIAL	COLOUR	SIZE
8917	MESH	WHITE WITH GREEN PIPING	ONE SIZE
8928	MESH WITH BUCKLE	WHITE WITH GREEN PIPING	SMALL/MEDIUM
8929	MESH WITH BUCKLE	WHITE WITH BLUE PIPING	LARGE/XLARGE
8932	POLYFOAM WITH BUCKLE	GREEN	SMALL/MEDIUM
8933	POLYFOAM WITH BUCKLE	BLUE	LARGE/XLARGE

### Adjustment

1. Install the pelvic belt, the part with the slots on the back of the chair and the part with the label FORMEDICA® on the seat.
2. Place the patient in the wheelchair or in the geriatric chair.
3. Raise the front of the belt.
4. Insert the belt forward in the slot in the rear part of the belt.
5. Wrap the front straps around the armrests of the chair or the back of geriatric chair.
6. Fasten securely and ensure that the ends do not reduce or impede the movement of the person's chair. Let the thickness of a hand between the waist of the patient and the contention.
7. Ensure that the patient is in a comfortable position. Check the condition of the skin at the groin regularly.

**Warning:** Use only while seated.





\* PLEASE NOTE THAT THIS INSTRUCTION SHEET IS GIVEN TO YOU FOR INFORMATIONAL PURPOSES ONLY AND IT IS THE RESPONSIBILITY OF THE ESTABLISHMENT TO ENSURE THAT THE METHOD OF USE OF THE PRODUCT IS SUITABLE FOR EACH SPECIFIC CIRCUMSTANCES OF THE USER.

## Security Belt

### Product description

The Security Belt can be used in a bed or wheelchair to prevent patients from sliding or falling. It allows the patient to roll from side to side.

### Characteristics

- Made of polyurethane foam laminated with cotton and a terry cloth interior
- Extra long belt made of polypropylene
- Height: 14.5 cm (5.75")



### Available sizes

CODE	DESCRIPTION	LENGTH	COLOUR	SIZE
7809	WITHOUT BUCKLE	58 cm (23")	GREEN	SMALL/MEDIUM
7810	WITH BUCKLE	58 cm (23")	GREEN	SMALL/MEDIUM
7811	WITHOUT BUCKLE	76 cm (30")	BLUE	LARGE/XLARGE
7812	WITH BUCKLE	76 cm (30")	BLUE	LARGE/XLARGE

### Adjustment

1. Place the patient in the wheelchair or in the geriatric chair.
2. Place the security belt on the patient. Wrap the front straps around the armrests of the chair or the back of geriatric chair.
3. Fasten securely and ensure that the ends do not reduce or impede the movement of the person's chair. Let the thickness of a hand between the waist of the patient and the contention.
4. Ensure that the patient is in a comfortable position. Check the condition of the skin at the groin regularly.

**Warning:** Use only while seated.

\* PLEASE NOTE THAT THIS INSTRUCTION SHEET IS GIVEN TO YOU FOR INFORMATIONAL PURPOSES ONLY AND IT IS THE RESPONSIBILITY OF THE ESTABLISHMENT TO ENSURE THAT THE METHOD OF USE OF THE PRODUCT IS SUITABLE FOR EACH SPECIFIC CIRCUMSTANCES OF THE USER.

## Walking Belt

### Product description

To assist caregiver in the safe transfer or ambulation of patients.

### Characteristics

- 3 strong polypropylene handles: 2 vertical (one on each side) and 1 h
- Made of polyurethane foam laminated with cotton and a terry cloth
- Velcro®-type closures and a safty buckle
- Height: 13 cm (5")



### Available sizes

CODE	LENGTH	COLOUR	SIZE
8980	64 cm - 110 cm (25" - 43")	GREEN	SMALL/MEDIUM
8981	110 cm - 155 cm (43" - 61")	BLUE	LARGE/XLARGE

\* PLEASE NOTE THAT THIS INSTRUCTION SHEET IS GIVEN TO YOU FOR INFORMATIONAL PURPOSES ONLY AND IT IS THE RESPONSIBILITY OF THE ESTABLISHMENT TO ENSURE THAT THE METHOD OF USE OF THE PRODUCT IS SUITABLE FOR EACH SPECIFIC CIRCUMSTANCES OF THE USER.

## Heel and Elbow Protectors

### Product description

The Heel and Elbow Protectors are used to prevent the formation of callosities or lesions caused by friction. They facilitate circulation by providing better heat and pressure distribution.



### Characteristics

- Velcro®-type closures
- Available in two types of material: synthetic fur (kodel) or polyurethane foam covered with brushed tricot and a terry cloth interior

### Available sizes

CODE	MATERIAL	COLOUR	SIZE
7642	KODEL	WHITE WITHOUT PIPING	ONE SIZE
7654	KODEL	WHITE WITH WHITE PIPING	SMALL/MEDIUM
7655	KODEL	WHITE WITH WHITE PIPING	LARGE/XLARGE
7656	FOAM RUBBER	GREY WITH WHITE PIPING	SMALL/MEDIUM
7657	FOAM RUBBER	GREY WITH WHITE PIPING	LARGE/XLARGE
6415	NYLON/LYCRA FABRIC MOLDED WITH A POLYMER GEL PAD	WHITE	SMALL/MEDIUM
6416	NYLON/LYCRA FABRIC MOLDED WITH A POLYMER GEL PAD	WHITE	LARGE/XLARGE



## Decubitus Pad

### Product description

Designed to prevent decubitus sores and to improve patient's comfort.

The pad can be placed directly against the skin. Improves circulation and diminishes pressure points.

### Characteristics

- Non-allergenic synthetic fur (kodel)
- Thickness: 13 mm (1/2")

### Available sizes

CODE	SIZE
7643	61 x 76 cm (24" x 30")
7644	76 x 102 cm (30" x 40")
7645	76 x 152 cm (30" x 60")



## Safety Mitts

### Product description

Recommended after vascular surgery and for bed-ridden patients.  
Fastened with straps (except child models).

### Available sizes

CODE	MATERIAL	MODEL	SIZE
8926	COTTON	CHILD	XSMALL
8927	COTTON	CHILD	SMALL
8921	POLYESTER	ADULT	ONE SIZE
8922	COTTON	ADULT	ONE SIZE

